

Defibrillator checks (every 1 to 3 months)

The defibrillator will self-test regularly. Check for any **visible or audible warnings** on the device that something is wrong.

The defibrillator should be in **standby-mode**.

Check that the defibrillator is **clean** and there is **no damage**.

Check the **battery indicator**.

Check that the **pads and batteries have not expired**.

Check the **cabinet** for damage.

Check that the defibrillator is **easily accessible** in an emergency.

Ready for use? Yes/No

Any issues:

Report to the Clerk – clerk@ludgvan.org.uk